DOMICILE - HOMECARE

ATLANTIS

CHAISE DE DOUCHE
SHOWER CHAIR
DUSCHSTUHL

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EN - Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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EN - Please read these safety instructions carefully before use and keep them for future question.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- Check carefully all parts of this item (screws, tubes, tips, grips)..
- Check regularly if all screws are tightened.
- Use only as a shower chair and in proper way : don't stand up on the chair.

- ALWAYS observe the weight limit on the labeling of your product.
- No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- DO NOT hang anything on the Chair.
- ALWAYS test to see that the chair and attachments are properly and securely locked in place before use.
- In case you didn't use your chair for a long time, let it checked by the technical staff, before use.
- Do not store outdoor, the chair must be used at home.
- Check the chair before use, in case of doubt ask your physician or provider.
- Use only the chair on a flat and solid ground.
- It is forbidden to seat on the armrests.
- Do not lean out of the chair to avoid losing balance.

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

The shower chair can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by enabling the person to sit while showering. Our shower chair is designed for use by people who are unable to stand to take a shower and must sit

instead. With the use of shower chairs, elderly people and those with mobility problems are able to have some independence by bathing themselves or having help doing so.

Indication

Used in the home when the patient is too debilitated to reach the bathroom or when the patient has reduced mobility and need to be seated while washing.

Contraindication

- Cognitive disorder - Loss of balance - Sitting impairment
- Patient weight is greater than the product's supported weight.
- Other contraindications: To be determined by the prescribing physician based on the patient's condition

Beware: The plastic shell seat has holes to drain water away from the seat. In very few cases, those drainage holes have the potential to entrap genitalia or rectal prolapse, leading to pain and serious injury such as cuts, lacerations and testicular bruising. If in doubt, please consult with your physician or retailer to determine a safe use of the device. You can also place material such as a towel over bath seats to prevent flesh or anatomy passing through.

3. INSTALLATION

A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

The chair can be used in the shower and under water. The chair is delivered ready to use. Castors are equipped with brakes.

Rental: this chair can be rented under the condition that it was correctly cleaned.

This user's manual belongs to that product and must be delivered to each new user.

4. CLEANING

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a dry cloth. It may be washed with soap and water or disinfected with non-chlorinated products.

Do not use any abrasive cleaners or high-pressure cleaners that could damage the product. Do not oil: slipping hazard.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See table page 6. Materials: The product is made up of aluminium tubing and a rigid plastic shell seat. Rear wheels with brakes. This product is biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions. The warranty does not cover mechanical, thermal, chemical or other damage resulting from insufficient or improper maintenance, storage or use of the product, as well as the use of non-original spare parts. The warranty does not cover items that wear naturally in accordance with their properties or intended use. Duration of use : 2 years.

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