DOMICILE - HOMECARE

DIFFUSION TABLE A MANGER BED TABLE BETTISCHE

MKT-H-NO-DIFFUSION-A

2025-03-12

IMPROVE THE NURSING LIFE*

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EN - Instructions for installation and use

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Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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4. NETTOYAGE, DESINFECTION, MAINTENANCE

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EN

- Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place. If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- ALWAYS consult with your physician or supplier to determine proper adjustment and use of the product.
- Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- Check carefully all parts of this item (screws, tubes, castors...).
- Check regularly if all parts are tightened.
- Use only in proper way : don't stand on the table.
- ALWAYS observe the weight limit on the labelling of your product.
- After assembling the table, make sure that it is securely locked in OPEN position and level to the ground before using.
- No modification must be made on this product. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- In case you didn't use your table for a long time, let it checked by the technical staff, before using.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear.

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Our overbed table is designed to fit over a bed or an armchair to offer the user a stable surface for eating, reading or having a support nearby.

Indications:

Relief of the caregiver when the patient is provided with a nursing bed or a bed frame, as well as predominant bed confinement of the patient; e.g., inability in independent feeding of the patient outside the bed.

Contraindications:

- mental illnesses like dementia, neurological diseases such as tetraplegia.
- Weight added to the tray greater than the product's supported weight.
- Other contraindications: to be determined by the prescribing physician based on the patient's condition

3. INSTALLATION

A wrong use or adjustment could lead to overstress of the product or of the patient. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Our bed table has a tilting foot and top for a functional and comfortable use in all situations, seating or lying down. The whole frame can be folded flat for storage. The laminated top is edged with an anti-shock border allowing books to be held. Supplied with 4 castors, two of them are lockable.

Side plates available for all models as an option.

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier or your authorized Herdegen distributor.

2. The bed table is delivered folded. To unfold it, pull out the locking pin located on the base and move the bed table upwards into a vertical position; release the pin to lock it. Make sure that the pin is fully engaged through the hole, see

(1).

3. To adjust the height of the plate, unscrew the knurled screw and lift the plate until the desired position is reached, then tighten the knurled screw by screwing clockwise, see (2). Beware, when unscrewed, hold the plate so that it doesn't fall.

4. To recline the plate, pull out the locking pin located under the plate and recline the plate at the desired position, then release the pin to lock it. Make sure that the pin is fully engaged through the hole, see (3).

5. To lock the brakes of the castors, see (4).

6. To add a side plate, recline the plate vertically and insert the side plate into the pipe, then lock it with the allen key and screw it (included), see (5). The side plate remains horizontal.

Beware: Do not lean on the suspended end of the table top; this may cause the table to tip. Do not exceed the maximum weight supported by the product.

Rental: This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

1- Visual inspection: ensure that the product has not been damaged and check for any defect.

2- Functional inspection: ensure that the product functions properly and repair it if needed.

3- Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories).

Do NOT use any abrasive cleaners or high-pressure cleaners that could damage the product.

Use only appropriate tools to perform the maintenance, cleaning and disinfection.

The user's manual is an integral part of the product and must be handed over to every new user.

4. CLEANING AND DISINFECTION

Clean the product regularly with a mild cleaning agent and a soft cloth.

Do not use any abrasive cleaners or high-pressure cleaners that could damage the product.

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See page 6. Biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions.

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